Pool lift: METALU S-400

Battery-powered pool lift.

Designed for in-ground pools.


METALU
ascensoracuático.com

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Installation of the lift.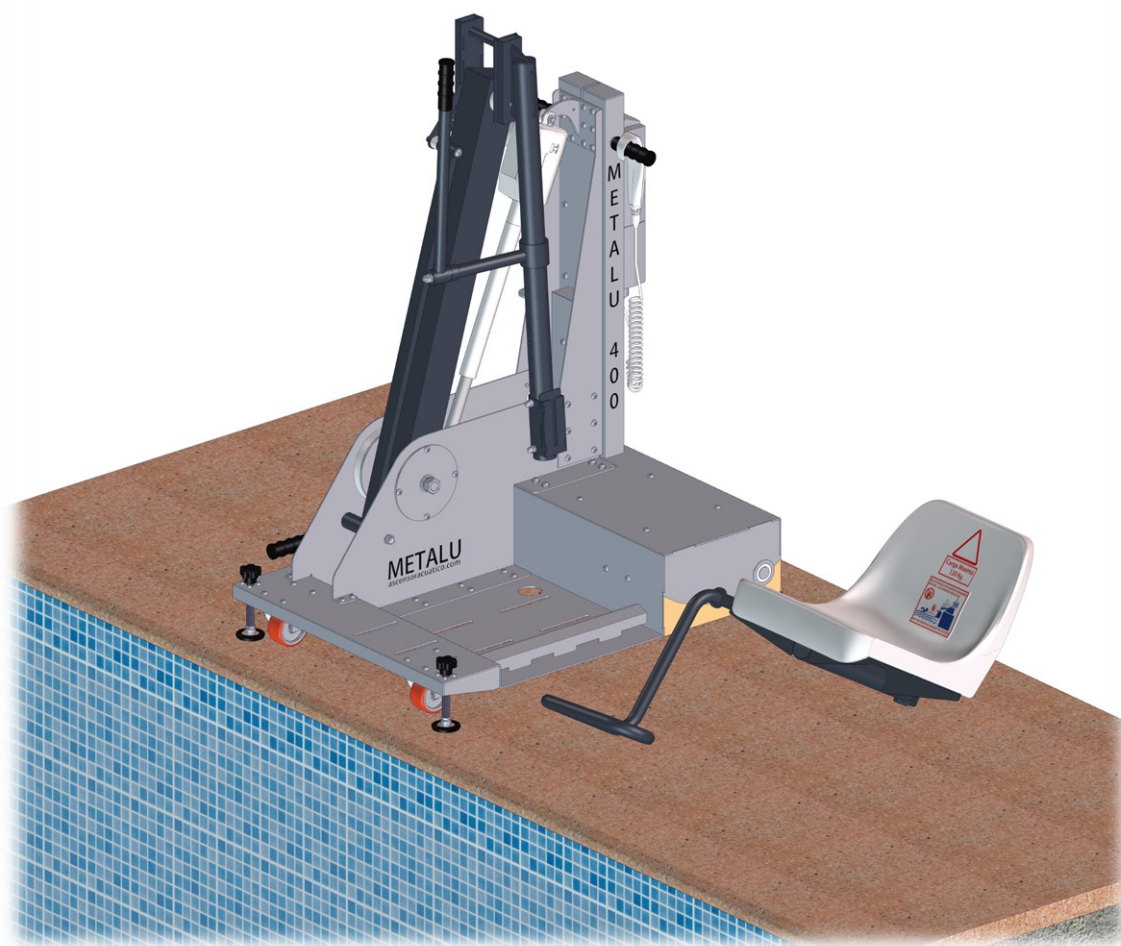

ISO 9001
ISO 14001
BUREAU VERITAS
Certification


Contents

1. Installation of the lift.	3
-----------------------------------	---


1. Installation of the lift.

Installing the METALU S-400 lift is very simple and no additional tools are needed. To correctly install the lift, follow the steps below (the regulations of each autonomous community should be considered during installation).

1. **Move the lift** to the place where it will be installed. Before moving it, make sure the front supports (screw pins) are fully raised. To facilitate transportation, we recommend moving it before installing the seat.
2. **Position the lift** where it is intended to be used, 50 to 120 mm from the edge of the pool. The front supports should not be resting on the overhang of the coping stone. Make sure the depth in this part of the pool is at least 0.9 m.

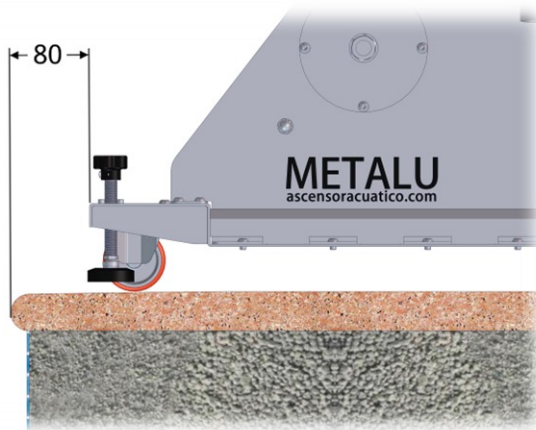
3. Place the **safety bar vertically**, such that it **rests in the hinge space** so that it actuates the micro-switch of the electrical system. The back supports shall end up below the level of the back wheels, leaving them suspended in the air.

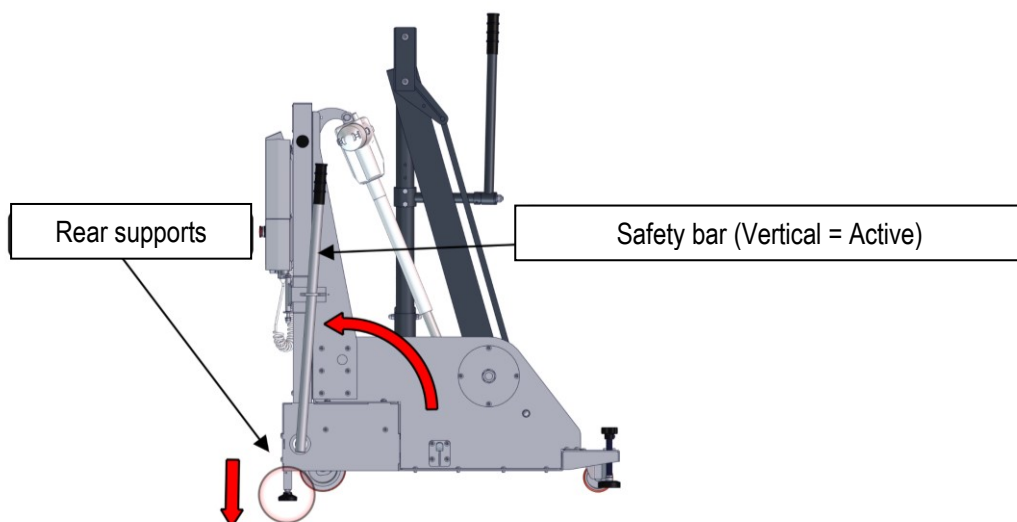4. **Lower the front supports** below the level of the front wheels, leaving them suspended in the air.

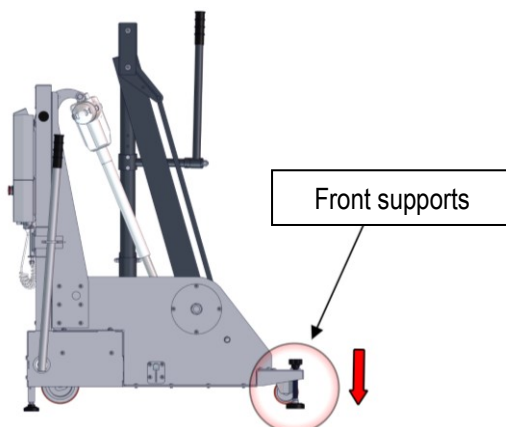5. **Place the seat** in the seat holder.

Once ready for use, the lift should look as it does in this image:

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